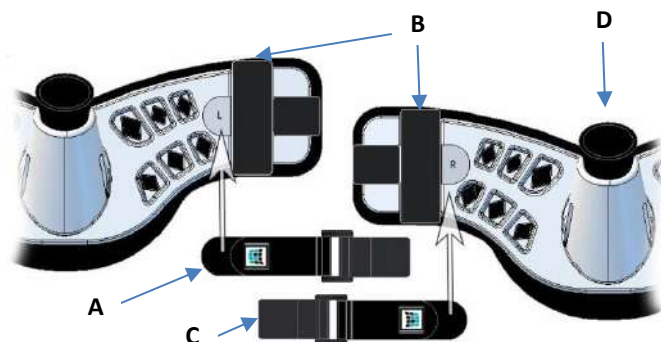
# IFU-HTSO

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## Hand-based Thumb Spica Orthosis

### INSTRUCTIONS FOR USE

**Before using the device, please read the following instructions completely and carefully. Correct application and care is vital to the proper functioning of the device**

#### INTENDED USE / INDICATIONS:

- Treatment of ulnar collateral ligament injury or to provide support and immobilisation in the treatment of soft tissue injuries
- Immobilization and stabilization of the first MCP joint.

#### CONTRAINDICATIONS:

Simple, complex, compound and/or comminuted fractures. Not for treatment of an open wound.

#### WARNINGS AND PRECAUTIONS:

- Device to be fitted by a qualified healthcare professional.
- Patients should consult their medical professional immediately if they experience any pain, swelling, sensation changes, or any unusual reactions while using this device.
- This Device should not be used on a patient with polyester or elastane allergy.
- Ensure the orthosis is not too hot prior to application.
- Do not apply over open wound.
- Instruct the patient on correct adjustment of the strap to maintain snug fit.
- Remove the device before any re-molding with a heat-gun, to avoid burning the patient.
- The orthosis will lose its shape above 57°C/135°F or under extreme sunlight. Cover with stockinet or clothing.
- Avoid water or vapor above 40°C/104°F for prolonged periods. Avoid heat sources (e.g. ovens, flames, sunny car windows, very humid conditions) for long periods.
- This device reduces mobility & dexterity. Do not operate heavy equipment.

#### 1. SELECT & GAUGE

- Select orthosis size based on sizing chart on package.
- Remove the backing paper from the round end (A) of the strap.
  - To treat Right-hand, press the round end of strap firmly into the "R" recess.
  - To treat Left-hand, press the round end of strap firmly into the "L" recess.

**2. HEAT** Lay the orthosis fabric side down with the thumb section flat on the FastForm rack, ensuring the plastic does not touch the sides. Heat the orthosis until it becomes soft / malleable. If not fully malleable, re-heat in 1-minute increments.

DO NOT heat for a total time greater than one-hour.

Convection ovens (preheated) = 5 -10 minutes @ to 75-85°C (167-185°F).

Nuwave ProPlus/Elite Ovens = 5 mins @ 220°F (105°C).

Nuwave Primo = 5 mins @ 260°F.

The HTSO is not suitable for heating in a waterbath.

For more detail see instructions on heating device/or QR Code.

#### 3. APPLY

- Place the HTSO over the thumb.
- Remove the 2<sup>nd</sup> backing paper (C) from the strap.
- Gently stretch the palmar wing across the palm and overlap on the back of the hand. Press the adhesive tab (C) into the soft Polytrex®.
- Confirm good alignment and positioning of the device particularly at 5<sup>th</sup> metacarpal and the palmar crease.
- Roll back the thumb liner (D) to form a comfort edge.

#### 4. MOLD

- Wrap an elastic bandage (not included) firmly but not tightly to conform the entire orthosis. A damp bandage accelerates cooling.
- Manipulate the device into the most appropriate position for healing.
- Rest patient's arm on a table until the orthosis is rigid then remove the elastic bandage.

#### 5. ADJUSTMENT

- Once the orthosis is set, articulate both hinges (B) for easier removal.

Note: the physician should advise the patient on how to adjust the strap for a snug fit.

Note: After removal the orthosis can be reformed with a heat gun in a localised area or in an oven. See [www.FastFormMedical.com](http://www.FastFormMedical.com) for further details.

#### GETTING IT WET

If allowed by the physician, it is acceptable for the orthosis to become wet during use. Note: It may take a number of hours to become completely dry. If allowed by the physician to loosen or remove the orthosis, the patient may dry it using a hair dryer on a cool setting.

#### CLEANING INSTRUCTIONS:

To clean the orthosis, rinse it in warm water. Note: it is not recommended to use household cleaners or personal hygiene products directly on the orthosis.

#### STORAGE:

This device is not sterile and should be stored between 4 - 35°C (39-95°F). It must also be protected from strong direct sunlight.

**WARRANTY:** FastForm Research Ltd covers defects in workmanship and materials for up to 12 weeks after this device is dispensed. Damage caused by normal wear & tear, abuse or alterations is not covered.

For further details on heating & application method, see:

[www.FastFormmedical.com](http://www.FastFormmedical.com) or [www.breq.com/fastform/HeatingTimes](http://www.breq.com/fastform/HeatingTimes)

-Custom moulded for single patient use.

-Not made with natural rubber latex -Rx only

**Notice:** while every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this

device. If a serious incident occurs with this device, please report it to the Manufacturer and your Competent Authority.

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**Breg, Inc.,**